

BioMed- Creatinine



Fixed rate

REF:	CRE106100	(2x50 ml)
	CRE106120	(2x60 ml)
	CRE106240	(2x120 ml)

INTENDED FOR USE

For the quantitative determination of creatinine in serum, plasma and urine

PRINCIPLE:

Creatinine reacts with picric acid in alkaline conditions to form a yellow-orange color complex.

The rate of formation of color is proportional to the creatinine quantity in the sample.

SPECIMEN COLLECTION :

Serum, heparinized plasma and 24 hours urine.

Urine must be diluted 1:50 with physiological saline 0.9%.

Do not use hemolyzed samples.

Creatinine in serum or plasma is stable for 1 day at room temperature (+15-25°C) and up to 7 days if in refrigerator at (+2-8°C).

Shake and bring the samples at room temperature (+15-25°C) before using

REAGENT COMPOSITIONS

R1	Creatinine standard	2.0 mg/dl
R2	Picric acid	38 mmol/l
R3	Sodium hydroxide	0.4 mmol/l

PACKAGE: Collection & storage.

Store at room temperature(+15-25°C).

Stable until the expiration date indicated on the label.

After the unsealing and the taking of the reagent ,it is advised to close up the bottle immediately in order to avoid evaporation, direct light exposure and bacterial contamination.

PRECAUTIONS & WARNING

AVOID PIPETTING WITH MOUTH

The Reagent (B) contains **sodium hydroxide** and , according to current regulation, is classified as: **C – Corrosive**

R34 - Causes burns.

S26 - In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

S45 - In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

S36/37/39 - Wear suitable protective clothing, gloves and eye/face protection.

A safety and precaution form is available on request.

The total concentration of non active components (preservatives, detergents, stabilizers) is below the minimum required for citation.

Anyway handle with care, avoid ingestion, avoid contact with eyes, skin and mucous membranes. The samples must be handle as potentially infected from HIV or Hepatitis

REAGENT PREPARATION & STABILITY :

Reagent (R2) is limpid/yellow; Reagent (R3) is limpid/colorless.

Mix Reagents (R2) and (R3) in the ratio 1+1

Working reagent is stable **2** days at room temperature

Use as much reagent quantity as necessary for the number of analyses to run.

REQUIRED MATERIALS NOT PROVIDED:

General Laboratory Equipment and instrumentations.

PROCEDURE:

Wavelength:	492 nm (490-500)
Optical path:	1 cm Light path
Temperature:	25C
Reading:	Against distilled water
Assay Type:	Increasing Kinetic

Pipetting in cuvette :

	SAMPLE	STANDARD
WORKING REAGENT	1000 µL	1000 µL
Sample	100 µL	
Standard		100 µL

Mix, and after 30 sec. read the absorbance A1 of the standard or specimen. Exactly 2 min. later, read absorbance A2 of standard or specimen.

CALCULATION:

A2-A1= A specimen or A standard

SERUM: Creatinine mg/dl = A specimen /A standard × 2

URINE(1:50): Creatinine mg/dl= $\frac{A \text{ specimen}}{A \text{ standard}} \times 2 \times 50$
Creatinine urine (mg/dl) × Vol. urine/24h in ml

Clearance creatinine (ml/min) = $\frac{\text{Creatinine urine (mg/dl)} \times \text{Vol. urine/24h in ml}}{\text{Creatinine serum (mg/dl)} \times 1440}$

EXPECTED VALUES:

Serum	
Men	0.7 - 1.4 mg/dl (0.062 - 0.124 mmol/L)
Women	0.5 - 1.2 mg/dl (0.044 - 0.106 mmol/L)
Children < 2 years old	0.3 - 0.6 mg/dl (0.027 - 0.053 mmol/L)

Urine:	up to 13.3 mmol/24h (1.5 g/24h)
---------------	---------------------------------

Clearance Creatinine	
Men	98 -160 ml/min
Women	95 -150 ml/min

The above mentioned values are to be considered as a reference.

It is strongly recommended that each laboratory establish its own normal range according to its geographic area, according to IFCC protocol.

WASTE DISPOSAL:

The disposal of the product must be in accordance with local regulation concerning waste disposal.

QUALITY CONTROL:

It is recommended to execute the quality control at every kit utilization to verify that values are within the reference range indicated by the methodology.

PERFORMANCE :

MEASURE INTERVAL LINEARITY:	0.1-20 mg/dL
DETECTION LIMIT(2 DS):	0.09mg/dL
SENSITIVITY:	1mg/dL = 0.046A

INTRA-ASSAY PRECISION: n=30

MEDIUM LEVEL	M=1.2mg/dL	C.V.=1.6%
HIGH LEVEL	M=3.6mg/dL	C.V.=1.07%

INTER-ASSAY PRECISION: n=30

MEDIUM LEVEL	M=1.8mg/dL	C.V.=2.15%
HIGH LEVEL	M=8.1mg/dL	C.V.=2.55%
INTER. ANALYZED	0.16-32mg/dL	
CORRELATION	r = 0.9879	n= 60
LIN.REGRESSION	y = 0.9999x - 0.1028	n= 60

INTERFERENCES:

Interferences are negligible up to:			
Glucose	500	mg/dl	Bilirubin > 55mg/dL
Urea	1.0	g/dl	Increase the reading
Ascorbic Acid	100	mg/dl	Hemoglobin > 100mg/dL
			Increase the reading

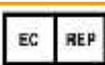
METHOD LIMITATIONS:

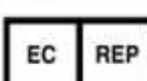
For concentration higher than 20 mg/dl, repeat the measure on a sample diluted 1:2 with saline solution and multiply the results by 2.

For a thorough evaluation of the interfering substances, consult: Young, D.S., et al., Clin.Chem. 21:1D (1975).

REFERENCES:

1. Jaffè, M. Zischr Physiol Chem .10 (391), 1886.
2. Henry, R.J. Ed., Clinical Chemistry: Principles and Technics (2°Ed). Harper and Row, 1974.
3. Young D.S., et al. Clin Chem. 21 (286), 1975

	Consult Instructions for Use
	Caution, Consult accompanying Documents
	In Vitro Diagnostic Medical Device
	Temperature Limitation
	Manufacturer
	Authorized Representative in the European Community
	Catalogue Number
	Batch Code
	Use by

 <p>EGY- CHEM for lab technology</p> <p>Badr City, Industrial Area Piece 170 250 Fadan In East of Elrubaki, EGYPT Office Tel: +202 26236727 / +202 26236598 Factory Tel: +202 23108170 / +202 23108171 Fax: +202 26240986 www.egy-chem.com</p>	  <p>MDSS GmbH Schiffgraben 41 30175 Hannover, Germany</p>
--	---